

RESPIRONICS TOTAL™ FACE MASK



A TOTALLY UNIQUE INTERFACE SOLUTION FOR NONINVASIVE VENTILATION AND CPAP THERAPY

The Respironics Total Face Mask is a unique new alternative for patients in the hospital or home who have difficulty tolerating nasal masks using Respironics BiPAP[®] or CPAP systems. It provides a fast, effective seal with no fitting and eliminates pressure sores caused by some nasal masks. As part of our Power Programs[™] for Managing Disease, the Total Face Mask will help secure a win in your battle to provide better patient care.

RESPIRONICS TOTAL™ FACE MASK


The Total Face Mask is an interface for adult patients using Respironics BiPAP or CPAP systems in the hospital or home. It is a comfortable alternative for patients who may not be able to obtain a good seal with a nasal mask, experience skin breakdown, or who are claustrophobic.

Because the Total Face Mask covers the entire face, there are no pressure points around the nose to cause sores or skin breakdown. An effective seal is created around the outside of the face as air pressure from the system inflates the soft, flexible sealing layer. This minimizes leaks while providing optimal airflow through the nasal passages. And, because the air pressure is able to circulate throughout the mask, it allows more natural, comfortable breathing for the patient.

Made of clear plastic, the lightweight faceplate adds to the comfort of the Total Face Mask. It does not obstruct the patient's vision, which helps to minimize the feelings of claustrophobia that some patients experience.

The built-in exhalation device can be used in conjunction with disposable circuits to provide an additional method of exhausting CO₂.

The Total Face Mask comes in one size with easy-to-use headgear that can be quickly fitted on most patients – an important advantage in critical care situations. Built-in safety features, such as an entrainment valve and quick release cord, give the care provider peace of mind.

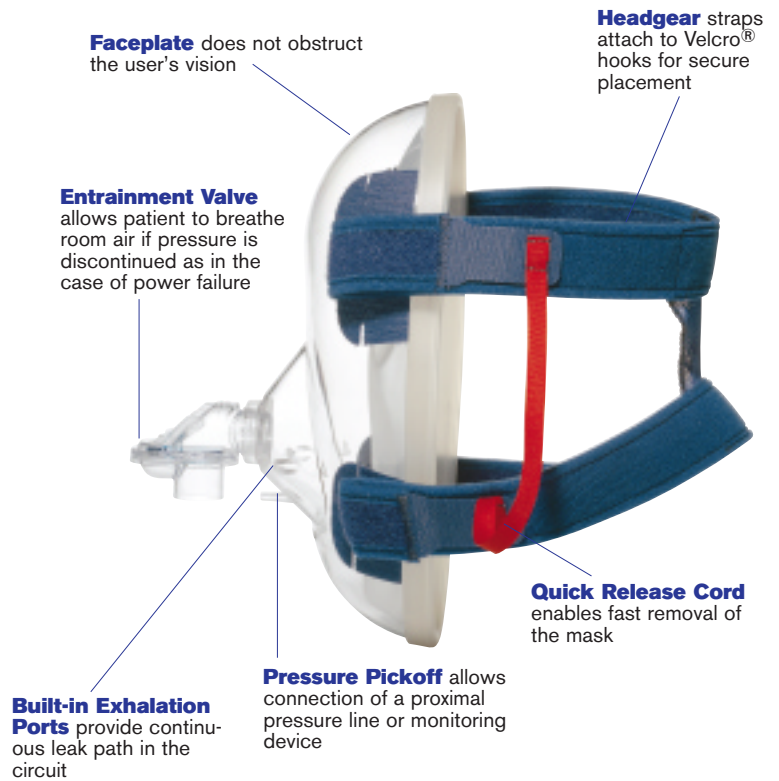

Respironics Total Face Mask

comes with comfortable, easy-to-use headgear. Sturdy cloth side straps and a mesh crown strap keep the mask evenly and securely in place and help to prevent mask leaks. Adjustable Velcro tabs on the straps attach to hooks on the faceplate of the mask for stable placement.



ORDERING INFORMATION Part Number

Total Face Mask with Headgear 302433



Global Headquarters:
Pittsburgh, Pennsylvania USA • 412-731-2100
Corporate Customer Service: 1-800-345-6443
Respironics Europe: +33-(0)1-55-60-19-80
Respironics Asia Pacific: +852-234-18

Covered by one or more of the following: U.S. Patent Nos. 5,655,527; 5,647,355; and 5,438,981.

Single Patient Use. Latex-free. Respironics Total, BiPAP and Power Programs are trademarks of Respironics, Inc. Velcro is a registered trademark of Velcro Industries. Caution: U.S. federal law restricts this device to sale by or on the order of a physician. © 2000 Respironics, Inc.

1004274
KW 3/28/00



SLEEP MANAGEMENT PROGRAM



CHRONIC OBSTRUCTIVE
PULMONARY DISEASE PROGRAM



RESPIRATORY INSUFFICIENCY
& FAILURE MANAGEMENT PROGRAM



ASTHMA AND ALLERGY PROGRAM



INFANT CARE PROGRAM